

Quick Start Guide CPR Module for REALITi 360

1. Power

- To Turn 'ON' the Device
Press and hold the power button for 1 second until the green PWR light turns on.
- To Turn 'OFF' the Device
Press and hold the power button for at least 3 seconds until all indicator lights turn off.

2. Bluetooth

- Blinking Blue Light
The device is attempting to connect via Bluetooth to the REALITi system.
- Solid Blue Light
The device is successfully connected to the REALITi system.

3. Battery Status Indicators

- White Light Device is charging.
- Green Light Battery is fully charged.
- Yellow Light Battery is at 50% charge.
- Red Light Battery is below 10% and requires charging.

How to Use with REALITi 360

1. Turn On the CPR Module
Press and hold the power button until the device powers on.
2. Open the CPR Connection Menu
 - Make sure the Monitor tablet's Bluetooth is activated to find nearby devices.
 - Ensure the monitor screen is running in REALITi app.
 - Swipe in from the right side of the screen to open the CPR connection menu.
3. Connect the CPR Sensor
 - Look for 'CPR Sensor' in the list of devices.
 - If you don't see it, press 'Rescan Devices' to refresh the list.
4. Disconnect the CPR Sensor
 - To disconnect, press the 'Disconnect' button in the CPR connection menu.

